

# Stone separating agent

## Indication

SHERASEP-G separates stone against stone and seals the surfaces simultaneously.

## General working instrucitons

- 1. Shake well before use.
- 2. Never put the brush into the original container.

## Stone against stone

1. Paint the part which should be insulated with SHERASEP-G, let it dry well. If the layer is too thick, you can blow it off (dry it) carefully! An insulating layer which is too thick may cause soft and inhomogeneous faces during the setting of the fresh stone.

### Warranty

SHERA Werkstoff-Technologie GmbH & Co. KG is certified according to DIN EN ISO13485 and guarantees for the products, due to a thorough quality control system, a flawless quality of its products. Our instructions for use are based on the results of our test laboratory. The technical data given can only be guaranteed if the processing is carried out as mentioned. The user is self-responsible for processing of the products. We are not liable for faulty results as SHREA has no influence on the processing. Nevertheless possibly arising claims for damages relate to the value of the products only.



SHERA Werkstoff-Technologie GmbH & Co. KG Espohlstrasse 53 · 49448 Lemförde · Germany Tel.: +49 (0) 54 43 - 99 33 - 0 · Fax: +49 (0) 54 43 - 99 33 - 100 www.shera.de · info@shera.de