# INSTRUCTIONS FOR USE SHERASEPAL-U



Colourless universal wax-based separating agent

#### 1. Indication

- Separates e.g. stone/plaster against stone/plaster, polyether impressions against SHERAPOLAN (model resin), SHERAPOLAN against SHERAPOLAN
- Does not separate stone/plaster against wax and stone/plaster against acrylic.

## 2. General processing advices

- Shake well before use.
- Spray a thin and even separating layer from a distance of 20 30 cm.
- Let the separating layer dry well

## 3. Recommendations and hints

- An insulating layer which is too think when insulating stone/plaster against stone/plaster may cause soft and inhomogeneous surfaces during the setting of the fresh stone/plaster. Therefore an isolation layer which is too thick can carefully be blown off (dry it).
- When insulating impressions of polyether against SHERAPOLAN the isolation should be applied by a brush. Spray SHERASEPAL-U into a container and isolate the parts which should be separated by using a brush. Let it dry well and repeat this process a second or a third time.

#### Warranty

SHERA Werkstoff-Technologie GmbH & Co. KG is certified according to DIN EN ISO 13485 and guarantees for the products, due to a thorough quality control system, a flawless quality of its products. Our instructions for use are based on the results of our test laboratory. The technical data given can only be guaranteed if the processing is carried out as mentioned. The user is self-responsible for processing of the products. We are not liable for faulty results as SHREA has no influence on the processing. Nevertheless possibly arising claims for damages relate to the value of the products only.

