



FRAME CLEANER

Indication

Ready for use cleaning and grease removal liquid for sandblasting frames before ceramic burning.

Preparation

- Fill GERÜSTCLEANER into a lockable container made of glass or acrylic.
- Put in frame to be cleaned.
- Lock and clean for 5 minutes in an ultrasonic bath.
- Leave the frame for further 5 minutes in the GERÜSTCLEANER.
- Pick up the frame only with tweezers.
- Avoid a whirl up of the liquid while taking out the frame.
- Do not wash the frame afterwards with water and never use compressed air, as there are always impurities in the compressed air.
- In case of frequent use the liquid should be renewed daily and the container should be washed.

Warranty

SHERA Werkstoff-Technologie GmbH & Co. KG is certified according to DIN EN ISO13485 and guarantees for the products, due to a thorough quality control system, a flawless quality of its products. Our instructions for use are based on the results of our test laboratory. The technical data given can only be guaranteed if the processing is carried out as mentioned. The user is self-responsible for processing of the products. We are not liable for faulty results as SHREA has no influence on the processing. Nevertheless possibly arising claims for damages relate to the value of the products only.

