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1. Indikation

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INSTRUCTIONS FOR USE

1. Indication

Ready-to-use isolating agent for 3D printed resin-models – e.g. against orthodontic & prosthetic resins (cold polymer).

2. Processing details:

Preparation:

- 3D Resins must be fully post cured, cleaned, and dry. (Incomplete polymerized 3D resins may impair the isolating effect.)

- **Before application of the isolating agent, block out undercuts with wax.** The isolating agent also sticks to the wax.

3. Application:

- Apply the isolating agent once, using either a natural or synthetic brush
- Coat SHERAiso-3d directly onto the model and apply it a bit thicker (especially in filament models), afterwards the isolation can be coated into a uniform layer. This procedure does not leave any uninsulated areas, the isolation agent does not dry as quickly when applied and prevents the dissolution of dried layers. Surplus can be wiped into a paper towel.
- Disperse any puddles (e.g. in deep fissures)
- Leave it to dry completely.

Advises and tips:

- Keep the bottle closed tightly after use.
- To avoid contamination and depositing dry isolation, we recommend applying directly onto the model or in a collecting vessel. Excess material must not be filled back in!
- (There is a risk that the isolation will gel and become unusable).
- A second layer is not necessary. In the case of a second application, the dried isolation layer may become brittle and lead to a deterioration of the surface. (Chequered appearance)
- The blue colour scheme allows a good visual inspection of the isolated areas and the layer thickness. The dried isolating area is easily recognizable.
- For models with large coating thicknesses or filament printers, first spray the blocked model with SHERASEPARAT and then let it dry. This prepares the surface perfectly for the SHERAiso-3d.

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