en



# SHERAprint-set p1

# INSTRUCTIONS FOR USE

## 1. Indication

Polishing set for the fast polishing of lightcuring 3D printing resins.

## 2. Application

#### Step 1:

Remove the coarse support structure with SCOTCH BRITE HS 25 mm coarse, colour brown.

#### Step 2:

Removal of further fine irregularities (rests of support structure) with SCOTCH BRITE HS 25 mm fine, colour violet.

Optionally, the object can be pre-polished at the polishing motor with SHERABIMSPASTE (REF 954301).

#### Step 3:

High gloss polishing with SHERALUX 711 polishing agent and COTTON NETTLE CLOTH. Here you can work with COTTON NETTLE CLOTH 80 mm for polishing motor or COTTON NETTLE CLOTH for the handpiece.

## 3. Disposal

Dispose of products in accordance with local regulations.

## 4. Guarantee

SHERA Werkstoff-Technologie GmbH is certified in accordance with EN ISO 13485 and guarantees flawless quality for its products thanks to an elaborate quality assurance system. Our user recommendations are based on so-called quide values determined in our test laboratory. These values can only be guaranteed if the specified process steps are adhered to. The user is responsible for the processing of the products. SHERA is not liable for faulty results, as SHERA has no influence on further processing. Any claims for damages that may nevertheless arise relate exclusively to the value of our products.

## Content

REF	Name	Quantity
910320	SCOTCH BRITE HS 25 mm coarse, colour brown	6 pieces
910319	SCOTCH BRITE HS 25 mm fine, colour violet	6 pieces
910401	COTTON NETTLE CLOTH 80 mm	1 piece
910305	COTTON NETTLE CLOTH HS 22 mm	6 pieces
954211	SHERALUX 711	100 g

All components can be reordered individually.

